
## JOULEMETER

## DEFIBRILLATOR TESTER UT- 3

Notice d'utilisation / User guide

V1.0 01.2007

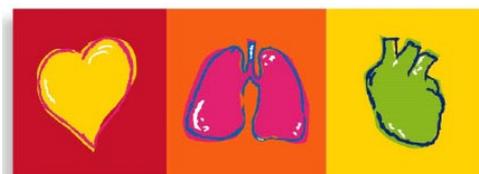The Art of Diagnostics

Part. No. 0-48-0088

### Sommaire

Page

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### Contents

<b>1</b>	<b>General points</b> .....	<b>15</b>
<b>2</b>	<b>Safety instructions</b> .....	<b>17</b>
2.1	Responsibilities of the user .....	17
2.2	Proper use .....	17
2.3	Organisational measures .....	17
2.4	Use in accordance with safety rules .....	17
2.5	Working with other devices .....	18
2.6	General safety instructions .....	18
2.7	Other specifications .....	18
2.7.1	Implicit permission .....	18
2.7.2	Guarantee conditions .....	19
2.8	Symbols .....	19
2.8.1	Symbols used in the manual .....	19
2.8.2	Symbols used on the device or on the accessories .....	19
<b>3</b>	<b>Intended use</b> .....	<b>20</b>
<b>4</b>	<b>Characteristics</b> .....	<b>20</b>
<b>5</b>	<b>Display and controls</b> .....	<b>21</b>
5.1	Front panel .....	21
5.2	Back panel .....	21
<b>6</b>	<b>Putting the device into operation</b> .....	<b>22</b>
6.1	Particular safety measures .....	22
6.2	Implementation of the measurement of energy .....	22
6.2.1	Defibrillation with the paddles .....	22
6.2.2	Defibrillation with a connecting cable .....	22
<b>7</b>	<b>Cleaning</b> .....	<b>23</b>
<b>8</b>	<b>Servicing and disposal at the end of the service life</b> .....	<b>23</b>
8.1	Servicing .....	23
8.2	Disposal at the end of the service life .....	23
<b>9</b>	<b>Specifications</b> .....	<b>23</b>
<b>10</b>	<b>Accessories</b> .....	<b>24</b>

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### 1 General points

- The **JOULEMETER UT-3** bears the CE mark in accordance with Directive 93/42/EEC relating to medical devices, and fulfils the fundamental requirements of Appendix I of that Directive.
- This product complies with the electromagnetic immunity requirements of standard EN 60601-1-2 "Electromagnetic Compatibility-Medical Electrical Equipment ».
- The safety of the user and trouble free device operation shall only be guaranteed when original SCHILLER parts are correctly used. Further, only the accessories mentioned in these instructions for use or accessories that have been expressly authorised by SCHILLER may be used. If the device becomes damp as a result of temperature variations, it may only be put back into service once it is completely dry.
- Devices become wet following variations in temperature, should only be used when they are completely dry.
- The manufacturer shall not be held liable for the safety, reliability and performance of the device if:
  - Assembly, extensions, adjustments, modifications or repairs are not performed by the manufacturer or by personnel authorised by the manufacturer.
  - The electrical installation of the premises of use does not comply with locally applicable regulations.
  - The device is not used in accordance with the instructions for use.
- The use of the device otherwise than described in the present manual is done under the whole responsibility of the user.
- The manufacturer may supply, on request, circuit diagrams, lists of components, descriptions, calibration instructions or any other information, for use by the qualified technical personnel of the user for repairing such parts of the device as have been stated to be repairable by the manufacturer. Such supply shall not in any case constitute permission or approval to modify or repair the device.
- This document describes a device type that complies with the safety standards applicable at the time of printing. SCHILLER reserves all rights to the circuits, processes, names, software and devices mentioned in these instructions for use.
- The user guide is an integral part of the device and should always be kept near the device. Close observance of the information given in the user guide is a prerequisite for using the device as intended for and correct operation and ensures operator safety. Therefore, be sure to read the complete user guide.
- The user guide informs the device operator about the intended use, exact function, operation and required preventive maintenance. It is not a substitute for a product training.
- The quality assurance process used by SCHILLER is in compliance with international standards ISO 9001 and ISO 13485.

- No part of these instructions for use may be reproduced without the written permission of SCHILLER.
- This device is not designed for any use other than that specifically described in this manual. Such use may be hazardous

### Manufacturer

SCHILLER Médical SAS  
4, rue Louis Pasteur – BP 90050  
F-67162 WISSEMBOURG Cedex  
Tel: +33 (0)3.88.63.36.00  
Fax: +33 (0)3.88.94.12.82  
E-mail: [info@schiller.fr](mailto:info@schiller.fr)  
Internet site: [www.schiller-medical.com](http://www.schiller-medical.com)

### Revision history

Version	Date	Comment
V1.0	January 2007	1st issue

## 2 Safety instructions

### 2.1 Responsibilities of the user

- This device may only be used by qualified persons with appropriate training.
- Make sure that the user has read and understood the instructions for use and particularly this "Safety instructions" section.
- Replace any faulty or missing parts immediately.
- The operator shall comply with the applicable regulations in the area of accident prevention and safety instructions.
- The device shall be stored out of the reach of children.
- Dispose of the packaging material and make sure that it remains out of the reach of children.

### 2.2 Proper use

- Always use the device in accordance with its stated technical specifications.
- Check that the mains specifications are in accordance with the specifications of the power supply.
- Do not use the device in an area with an explosion hazard or in an area with inflammable gases such as anaesthetic gases.

### 2.3 Organisational measures

- This device may only be used after explanations about its working and safety instructions are given and understood.
- Keep the instructions for use to hand in the place of use of the device. Make sure that they are always complete and legible.
- In addition to the instructions for use, follow the ordinary legal requirements and any other requirements applicable in the area of accident prevention and environment protection.

### 2.4 Use in accordance with safety rules

- Read and follow the instructions for use, particularly the safety instructions.
- Immediately report to the person responsible any change liable to affect safety, including changes relating to the behaviour of the device in operation.
- Ensure of the correct operation of the device before each use.

- Before starting the device, check that there is no damage to its housings.
- The device may only be exposed to wide temperature variations for a very short time. Very high temperature variations may lead to the formation of condensation water on the device. However, if condensation water does form, dry the device and all its connections.
- Install the device so that it cannot fall on the user or the floor.
- The device must not be connected nor used on a patient.

### 2.5 Working with other devices

- The **JOULEMETER UT-3** should only be connected to SCHILLER defibrillators.
- Use only the accessories supplied and recommended by SCHILLER. The use of accessories manufactured by third parties may lead to injuries, provide inaccurate information and/or lead to device damage.
- Verify that the paddles or the connecting cables do not present deteriorations or defects of insulation.
- Very high leakage currents (total leakage currents) can be a hazard for the user.
- The working of the device may be influenced by the presence of magnetic or electrical fields from X-ray machines, tomography machines, radio transmitters and mobile telephones, among other devices. Avoid the use of such devices and keep a sufficient distance from them.
- The **JOULEMETER UT-3** should not be used in MRI rooms.

### 2.6 General safety instructions

- Risk of injury for the user if the device is used with faulty housings or cables! That is why the following instruction must be followed strictly:  
Immediately replace any faulty device, cable or connector.

### 2.7 Other specifications

#### 2.7.1 Implicit permission

- The possession or acquisition of this device does not amount to express or implicit permission to use this device with parts which, alone or in combination with this device, may be covered by one or more patents.

### 2.7.2 Guarantee conditions

The **JOULEMETRE UT-3** from SCHILLER comes with a guarantee (from the date of purchase) covering all workmanship or material defects. The guarantee does not however cover damage due to negligence or improper use. The guarantee includes the replacement of faulty parts. It excludes any liability for consequential damage. The guarantee shall not apply if unauthorised or unqualified personnel repair the device.

In the event of a fault, the device in question shall be returned to the local SCHILLER representative or directly to the manufacturer. The manufacturer warrants the safety, reliability and proper operating of the device only if

- any assemblies, additions, new settings, modifications or repairs are carried out by persons approved by the manufacturer and
- the device and its accessories are used in accordance with the manufacturer's specifications.

### 2.8 Symbols

#### 2.8.1 Symbols used in the manual

The device meets the essential requirements of Council Directive 93/42/CEE about medical devices

#### 2.8.2 Symbols used on the device or on the accessories

The device meets the essential requirements of Council Directive 93/42/CEE about medical devices

Do not dispose with household waste. Dispose separately

Danger, High Voltage present during shock delivery.

See user's manual

Direct current

### 3 Intended use

- The **JOULEMETER UT-3** is intended to measure the delivered energies of all SCHILLER manual and automated defibrillators.

### 4 Characteristics

- The **JOULEMETER UT-3** can measure the energy of monophasic or biphasic defibrillation shocks
- The **JOULEMETER UT-3** generates a physiological fibrillation signal which is recognised by all SCHILLER automated defibrillators as a shockable rhythm.
- The energy of defibrillation can be transmitted to the **JOULEMETER UT-3** via defibrillation paddles for adults or by connecting the defibrillator to the Joulemeter via a cable provided with 4 mm male banana. In this case the defibrillation discs must be removed from the device.
- The **JOULEMETER UT-3** can measure energies of defibrillation without having to respect a polarity of connection.
- The **JOULEMETER UT-3** is only fed by the mains; it does not have an internal battery.
- The **JOULEMETER UT-3** is a measuring device which must be calibrated regularly.

The standard equipment of the **JOULEMETER UT-3** consists of:

- the device
- 2 discs for defibrillation with paddles
- one mains power supply pack
- the user's manual

### 5 Display and controls

#### 5.1 Front panel

- 1 LCD screen for displaying of the delivered energy and the number of shocks delivered since the starting of the device.
- 2 2 removable disks for defibrillation with paddles for adults.
- 3 Insulator of separation of the paddles.

#### 5.2 Back panel

- 4 Connector for calibration of the device. This is only possible in factory.
- 5 Connector for the power pack
- 6 On/Off switch

## 6 Putting the device into operation

### 6.1 Particular safety measures

#### Caution

- Never put the 2 paddles of the defibrillator of which you want to measure the delivered energies in contact one with the other. **Risk of damage of the defibrillator.**
- Verify that the paddles of the defibrillator cover completely the contact discs of the Joulemeter.
- Firmly apply the paddles to the discs of defibrillation of the Joulemeter before starting the loading of energy on the defibrillator.
- Keep the paddles applied on the discs until the delivered energy is displayed.

### 6.2 Implementation of the measurement of energy

#### 6.2.1 Defibrillation with the paddles

- Verify that the discs for defibrillation by paddles are well in place on the joulemeter.
- Connect the **JOULEMETER UT-3** to the mains through the power supply pack and switch the device on with the switch (6). The device displays the messages "**UT-3 Defi Tester**" then "**Ready...**" on the screen.
- Put the defibrillator on and select the energy to be measured.
- Firmly apply the paddles to the discs of the joulemeter by taking care that they entirely cover the discs.
- Charge the energy on the defibrillator.
- When the shock is ready, deliver it in the joulemeter by maintaining the paddles applied to the discs until the delivered energy is displayed.

#### 6.2.2 Defibrillation with a connecting cable

- Remove the discs (2) from the joulemeter and plug in the contacts the 4 mm banana type connectors of the connecting cable.
- Connect the cable to the defibrillator
- Connect the **JOULEMETER UT-3** to the mains through the power supply pack and switch the device on with the switch (6). The device displays the messages "**UT-3 Defi Tester**" then "**Ready...**" on the screen.
- Start the defibrillator and select the energy to be measured.
- Charge the energy on the defibrillator.
- Deliver the shock.

- Check the value of the energy displayed on the joulemeter.

Note : When using an Automated External Defibrillator, follow the instructions of the defibrillator after connecting the cable.

## 7 Cleaning

During cleaning, the power supply pack must be disconnected from the device. No liquid may be allowed to enter the device. If liquid does penetrate into the device, clean the device and verify it completely.

The outer surfaces of the device may be cleaned with cotton wool moistened with alcohol.

**Caution ! Risk of damage to the device. Never use phenol based cleaning agents or agents containing peroxide derivatives to disinfect the surfaces of the device housing.**

## 8 Servicing and disposal at the end of the service life

### 8.1 Servicing

The device must be calibrated in factory once a year.

### 8.2 Disposal at the end of the service life

At the end of its service life, the device and the accessories must be disposed of in compliance with the local regulations.

If you do not have a local mean for recycling, send the device back to the producer.

Improper disposal harms the environment and human health due to the presence of dangerous substances in electrical and electronic equipment.

## 9 Specifications

- The device is calibrated at delivery. After one year of use, a new calibration must be carried out.
- Precision of the displayed values: +/- 5%
- Maximum input voltage: +/- 5 kV
- Maximum frequency of the shocks acceptable: 1 shock at 360 Joules per minute or 2 shocks at 180 Joules per minute
- Power supply pack:
  - Europe: 230V 50Hz 10VA
  - U.K.: 230V 50Hz 10VA
  - US: 100-240V 50-60Hz 5VA

### Physical and environment requirements

- Transport /storage:

Temperature: – 20 to + 50 °C

Relative humidity rate: 30 to 95 %, without condensation

Atmospheric pressure: 500 to 1060 hPa

- Operation:

Temperature: 0 to + 40 °C

Relative humidity rate: 30 to 95 %, without condensation

Atmospheric pressure: 700 to 1060 hPa

### Dimensions and weight

Width 255 mm

Depth 205 mm

Height 107 mm

Weight 1,1 kg without power supply pack, 1,6 kg with power supply pack

## 10 Accessories

Part Number	Designation
1-036-0000	<b>JOULEMETER UT-3</b> with discs for paddles
3-55-0021	Power supply pack Europe
3-55-0023	Power supply pack UK
3-55-0024	Power supply pack US